

D15™



FILTER SOLUTIONS

How to Use the D15™ Filter Face Mask

Wearing a D15™ Filter Face Mask is not a primary preventative measure and should not provide a false sense of protection that leads to a misuse of masks. The mask should always be used in conjunction with WHO COVID-19 protocols for infection prevention.

Always use a new or disinfected D15™ Filter Face Mask or Filter Element. See washing and disinfecting instructions below.

Putting on your D15™ Filter Face Mask

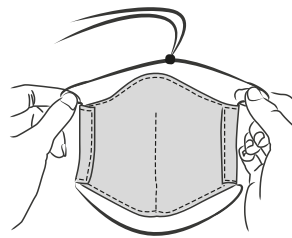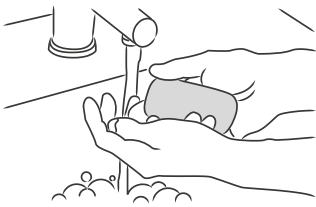

1. Take your D15™ Filter Face Mask out of the packaging and disinfect as per the instructions.
2. Wash your hands with alcohol-based hand rub or soap and water.

1. *Thata i-mask yakho ngaphandle, yihlanze njengemiyalelo.*
2. *Washa izandla ngensipho namanzi.*


3. Pick up your mask by touching the elasticated cord only; avoid touching the mask itself.
4. Fit elasticated cords around head.
5. Fit the mask around your mouth,

3. *Thinta intambo yelastiki kuphela, ungathinti i-mask uqobo.*
4. *Linganisa intambo ekhanda.*

- nose and chin, ensuring that there are no gaps.
6. Adjust by touching only top or the elastic to make it tighter around your head.

5. *Faka Kahle i-mask izungeze umlomo, ikhala nesilevu, qinisekisa asikho isikhala.*
6. *Lungisa ngentambo ukuqinisa ekhanda.*

While wearing your D15™ Filter Face Mask

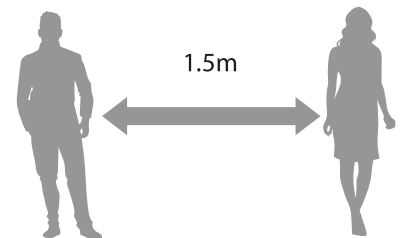

1. Your mask should be completely on or off – do not wear under your chin.
2. Never wear your mask inside-out.
3. Tighten the toggle system to fit around your head comfortably.

1. *I-mask isilungile, ungayigqoki phansi kwesilevu.*
2. *Ayihlanekezelwa.*
3. *Qinisa inkinobho ukulinganisa ekhanda.*

4. Remove your mask if it gets damp or soiled.
5. Do not touch or adjust your mask while it is on and avoid touching your face.

4. *Khipha i-mask uma imanzi noma ingcolile.*
5. *Ungathinti i-mask uma uyigqokile, ungathinti ubuso bakho.*

6. If you touch your mask, wash your hands immediately with alcohol-based hand rub or soap and water.
7. Continue following social distancing and good hygiene practices while wearing your mask.

6. *Uma uthinte i-mask, geza izandla ngokushesha.*
7. *Qhubeka uziqhelise kubantu uhlanzeke uma usagqoke i-mask.*

Removing your D15™ Filter Face Mask




1. Hold the elastic cords only and loosen the toggle system from your face.

1. *Ukukhiphai-mask, bamba intambo.*



5 Minutes

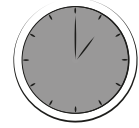

2. Disinfect your mask in hot water immediately as per the washing and disinfecting instructions.


2. *Hlanza i-mask emanzini ashisayo.*

Washing and disinfecting your D15™ Filter Face Mask



1. Place your mask with filter in a bowl and cover with boiling water.
2. Leave the complete mask in the hot water for at least 5 minutes to disinfect.
3. Wash your hands using hand-washing protocols for COVID-19.
4. Wash your face using COVID-19 protocols.

1. *Cwilisa imask i-filter emanzini abilayo.*
2. *Faka imask emanzini ashisayo imizuzu emihlanu.*
3. *Geza izandla.*
4. *Geza ubuso bakho usebenzisa izimiso ze-COVID019.*

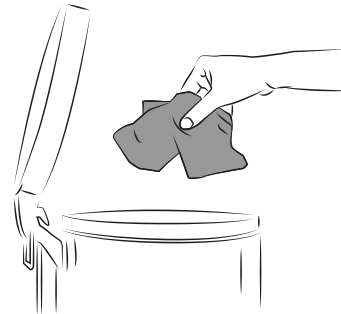

5. If the mask is dirty, remove the filter and hand-wash the mask, or machine-wash on a gentle cycle.
6. Do not squeeze, wring or hand-wash the filter, as the delicate structure may be damaged.
7. Leave the mask and filter to air-dry. When dry, insert the filter

5. *Uma i-mask ingcolile, khipha ifilter uwashe i-mask.*
6. *Khipha ifilter kwi-mask, uyomise.*
7. *Yomisa i-mask ne-filter, uma sekomile usungabuyisela i-filter kwi-mask.*

back into the mask – it is now ready to wear.

8. Once the filter has been used and disinfected 7 times, discard after disinfecting and insert a new filter.
9. If you cannot disinfect your mask or filter before disposal, treat as hazardous waste.

8. *Lahla ifilter emva kokuyihlanza kayi-7.*
9. *Uma ungayihlanzanga i-mask okanye ifilter, kuthathe loko njengodoti oyingozi.*

Scan or click the below QR codes for more information on our D15™ product range



D15™ Industrial II Filter Face Mask



D15™ Corporate Filter Face Mask



D15™ DIY Filter Face Mask Kit



D15™ Filter

Powered by
brits
nonwoven

www.brits.co.za